**INSTRUCTIONS FOR USE**

CO2 INSUFFLATION TUBING WITH HYDROPHOBIC FILTER

-  **CAUTION**
-  **TEMPERATURE LIMITS**
-  **KEEP AWAY FROM DIRECT SUNLIGHT**
-  **PROTECT FROM MOISTURE**
-  **LOT CODE**
-  **QUANTITY**
-  **EXPIRY**
-  **DO NOT USE IF PACKAGING IS DAMAGED**
-  **MEDICAL DEVICE**
-  **DISPOSE OF AFTER 24 HOURS OF USE**
-  **NON-STERILE**
-  **MANUFACTURER**
-  **LATEX FREE**
-  **PHT FREE**
-  **READ INSTRUCTIONS FOR USE**
-  **REFERENCE**
-  **EC ACCORDING TO DIRECTIVE 93/42/EEC AND UPDATE 2007/47/EC**

-  • Using the tubing for more than 24 hours can cause bacteria to develop in the stagnant water within tubing.
- Before using this device, please read the instructions for the endoscope you will be using, as well as for any accessories that will be used.

**CO2 INSUFFLATION TUBING WITH HYDROPHOBIC FILTER**

Please read the following information carefully.

Failure to use the product as intended may have harmful consequences for the patient.

**Important reminder:**

This document contains instructions for using the CO2 tubing with hydrophobic filter: **“ENDOfube for Insufflation”**. It contains no references to particular medical procedures. The manufacturer will not be held liable for any problems caused by improper use of the device.

**I. INDICATION**

**ENDOfube for Insufflation** is a tube with a hydrophobic filter to supply the endoscope with carbon dioxide (CO2) through a CO2 insufflator during gastrointestinal endoscopy procedure.


**ENDOfube for Insufflation** is a device that can be used for 24 hours following removal from packaging.

One end of the **ENDOfube for Insufflation** connects to endoscopy insufflators fitted with a male or female Luer lock fitting, and the other connects to the **ENDOfube for Wash Bottle CO2's** 2- or 3-way valves, with its female Luer lock fitting.

Two types of **ENDOfube for Insufflation** are available:

- **TUBCO2LUER - CO2 INSUFFLATION TUBING WITH HYDROPHOBIC FILTER - LUER CONNECTION** is the tubing to be used with CO2 endoscopy insufflators with a male connector.
- **TUBCO2FUJI - CO2 INSUFFLATION TUBING WITH HYDROPHOBIC FILTER - ADAPTER CONNECTION** is the tubing to be used with Olympus UCR, FUJIFILM GW-1 and GW-100 insufflators, in combination with the following adaptors:
  - **04700** for the Olympus UCR,
  - **04628** for the FUJIFILM GW-1
  - **04712** for the FUJIFILM GW-100.

Part no.	Description
1	Male Luer lock fitting for 4.8mm x 8mm PVC tube
2	4.8mm x 8mm DEHT (DEHP free) PLASTICISED PVC TUBE
3	Standard membrane filter, outer diameter 53mm
4	4.8mm x 8mm DEHT (DEHP free) PLASTICISED PVC TUBE
5	Female Luer lock fitting for 4.8mm x 8mm PVC tube

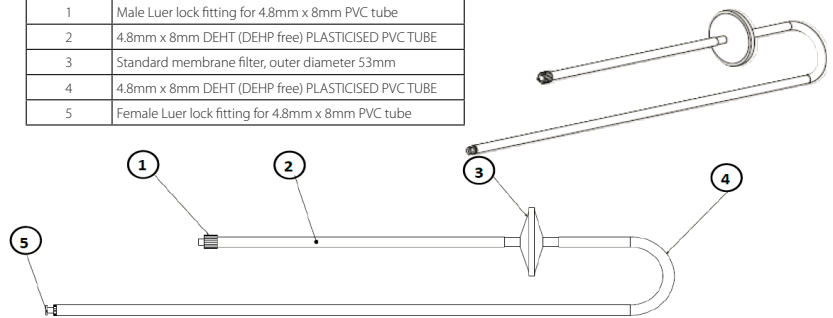**CONTRAINDICATIONS:**

The contraindications are those particular to any endoscopy procedure.

**II. STORAGE / PACKAGING**

**STORAGE**

- **Room temperature: +10°C to +50°C**
- **Expiry date: 2 years (see on the label)**

**PACKAGING**

**ENDOfube for Insufflation** is packaged individually in boxes of 20.

Once removed from its packaging, the tubing must be used within 24 hours of opening.

### III. WARNINGS AND PRECAUTIONS

#### The ENDOfube for Insufflation range

- Must be used within 24 hours of opening
- Must never be connected or reconnected to an endoscope that has not been washed as per manufacturer instructions.
- Must only be used by qualified endoscopy personnel.
- Not designed to be reused after being used for 24 hours.
- Must be disposed of as waste in compliance with the country's applicable regulations.

When changing the water bottle connected to an **ENDOfube for Wash Bottle + CO2** system, disconnecting the system between two patients, or during any other procedures, authorised personnel must use appropriate techniques to avoid contaminating the **ENDOfube for Insufflation** tubing connected to the **ENDOfube for Wash Bottle + CO2** system.  
Be sure to identify any leakages in the connector to avoid wetting the floor and causing staff to slip.

The **ENDOfube for Insufflation** comes with a **male Luer** lock fitting. In normal conditions of use, the **ENDOfube for Insufflation** tube is designed to be used away from the patient and close to the endoscopy column.  
**Special care must be taken to avoid inadvertently connecting the connector to an existing line in the patient with a female connector.**

### IV. USE / MAINTENANCE / CLEANING / DISINFECTION / STERILISATION

#### USE:

It is important that you familiarise yourself with the device and complete a trial run before using it in a clinical context  
(See also the instructions for the endoscope and any accessories used)

1. Open a bottle of sterile water (25cl, 50cl, or 100cl).
2. Open the packaging and remove the ENDOfube for Wash Bottle + CO2 system.
3. Insert the tube into the bottle of sterile water and press the stop to make sure it is watertight.
4. Check that the clamp is in the open position.
5. Connect the air/water connector to the endoscope.
6. Open the packaging and remove the ENDOfube for Insufflation system.
7. Connect the ENDOfube for Insufflation tube to the ENDOfube for Wash Bottle + CO2 and then the insufflator.
8. Turn on the processor's light source and then turn on the insufflator.
9. If you are using air, check that the valve on the ENDOfube for Wash Bottle + CO2 tube is open so that air can move from the water bottle to the endoscope.
10. If using CO2, you must turn off the endoscope's air pump and check that the valve on the ENDOfube for Wash Bottle + CO2 tube is open so that the CO2 can move from the insufflator to the endoscope and water bottle.
11. Before starting the procedure, and irrespective of the procedure used (see points 9 and 10), check that it is working properly by starting the endoscope's air/water or air/CO2 canal. If there is a malfunction, check that the water bottle is properly sealed and that the tubes are properly connected to the various devices.
12. If you wish to change the source of insufflation (air or CO2) during the procedure, refer to points 9 and 10.
13. If you need to replace the water bottle, use suitable techniques.
14. Once the procedure is complete, clamp and unplug the ENDOfube for Wash Bottle + CO2 system.
15. You can then turn off the processor and insufflator once you have unplugged the ENDOfube for Wash Bottle + CO2 system.

Make sure that the device is properly disposed of within 24 hours of opening the packaging. A traceability label is provided for this purpose. Dispose of the device according to applicable regulations in your country.

#### CLEANING / DISINFECTION / STERILISATION:

It is a non-sterile device for 24-hour use that cannot be sterilized. Disinfection or cleaning is not necessary.

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